

Directions for Use



This calf puller is easy to handle.

The rump frame is placed over the cow's rear with the open end towards the backbone. The calf puller can not slip out of position. It leaves both hands free to assist cow and calf.

The ropes are attached to both of the calf's legs and to the lever mechanism in that order.

Now gently tension the ropes and it will adopt the correct position whether the cow is standing up or lying down.



During a normal presentation


One should draw downwards in the direction of the udder until the head is born and then in a horizontal line.

When the calf's pelvis threatens to lock in the cow's pelvis, one should draw more in the direction of the udder.



In the case of a posterior presentation

The draw should be upwards towards the tail until the legs are born and then change to a horizontal line

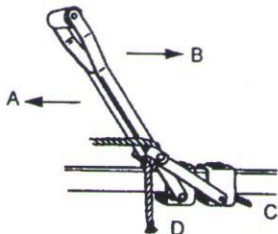


The quick release

Depress catch C and with the same hand, push the whole lever mechanism towards the cow.

The handle will swing forward and automatically release catch D.

Do not press the handle.

If the ropes are too taut, ease the handle backwards to allow catch C to be released more easily.

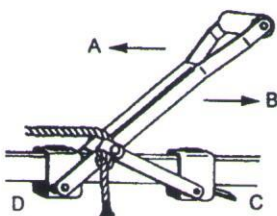

To make a tension free stroke

One should draw downwards in the direction of the udder until the head is born and then in a horizontal line.

When the calf's pelvis threatens to lock in the cow's pelvis, one should draw more in the direction of the udder.

A normal delivery requires a pull of 150-250kg. a stronger pull may cause damage to the cow, calf or apparatus. If the pole of the calf puller is moved more than 30 degrees sideways, it may be damaged. Always take care to ensure sufficient room around the cow during the delivery, this renders a sideways movement of the pole unnecessary.

The manufacturer does not accept responsibility for damage caused by the above mentioned or other improper use of the calf puller.